DO 2.1
DO 2.1-10

NÁVOD NA INŠTALÁCIU, OBSLUHU A ÚDRŽBU
INSTALLATION, OPERATION AND MAINTENANCE MANUAL
ИНСТРУКЦИЯ ПО УСТАНОВКЕ, ОБСЛУЖИВАНИЮ И УХОДУ
DIE ANLEITUNG ZUR INSTALLATION, BEDIENUNG UND WARTUNG
NOTICE D'INSTALLATION, DE MISE EN MARCHE ET D'ENTRETIEN
INSTRUKCJA INSTALACJI, OBSŁUGI ORAZ NAPRAWY BIEŻĄCEJ
NÁVOD K MONTÁŽI, OBSLUZE A ÚDRŽBĚ

SK

EN

RU

DE

FR

PL

CS

	<u>OBSAH..... 2</u>
	<u>CONTENTS..... 11</u>
	<u>СОДЕРЖАНИЕ..... 20</u>
	<u>INHALT..... 29</u>
	<u>SOMMAIRE..... 38</u>
	<u>SPIS TREŚCI..... 47</u>
	<u>OBSAH..... 56</u>

Ed. - 7

OBSAH

DÔLEŽITÉ INFORMÁCIE.....	3
1. OZNAČENIE CE.....	3
2. UPOZORNENIA.....	3
3. UPOZORNENIA A SYMBOLY.....	4
4. SKLADOVACIE A PREPRAVNÉ PODMIENKY.....	4
5. TECHNICKÉ ÚDAJE.....	5
6. POPIS VÝROBKU.....	6
7. POPIS FUNKCIE.....	6
INŠTALÁCIA.....	7
8. PODMIENKY POUŽITIA.....	7
9. INŠTALOVANIE VÝROBKU.....	7
10. SCHÉMA ZAPOJENIA.....	8
11. PRVÉ UVEDENIE DO PREVÁDZKY.....	8
OBSLUHA.....	9
12. ZAPNUTIE VÝROBKU.....	9
ÚDRŽBA.....	9
13. INTERVALY ÚDRŽBY.....	9
14. ÚDRŽBA.....	9
15. ODSTAVENIE.....	10
16. LIKVIDÁCIA PRÍSTROJA.....	10
17. INFORMÁCIE O OPRAVÁRENSKEJ SLUŽBE.....	10
18. VYHLÁDÁVANIE PORÚCH A ICH ODSTRÁNENIE.....	10
ROZSAH DODÁVKY.....	65

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1. OZNAČENIE CE

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CONTENTS

IMPORTANT INFORMATION	12
1. CE MARKING	12
2. WARNINGS	12
3. CAUTIONS AND SYMBOLS	13
4. STORAGE AND TRANSPORT CONDITIONS	13
5. TECHNICAL DATA	14
6. PRODUCT DESCRIPTION	15
7. DESCRIPTION OF FUNCTION	15
INSTALLATION	16
8. USE	16
9. INSTALLATION	16
10. WIRING DIAGRAM	17
11. FIRST OPERATION	17
OPERATION	18
12. SWITCHING SUCTION UNIT ON	18
MAINTENANCE	18
13. MAINTENANCE SCHEDULE	18
14. MAINTENANCE	18
15. STORAGE	19
16. DISPOSING OF THE APPLIANCE	19
17. REPAIR SERVICE	19
18. SOLVING PROBLEMS	19
PARTS LIST	65

IMPORTANT INFORMATION

1. CE MARKING

Products labeled with the compliance mark **CE** meet safety Guidelines of the European Union MDD 93/42/EEC.

2. WARNINGS

2.1. General warnings

- The installation, operation and maintenance manual is an integral part of the appliance and must be kept with it. Exact observance of this manual is the prerequisite of the correct use and operation according to model.
- For the safety of operating personnel and trouble-free operation of the appliance, we strongly advise the use and fitting of original parts and accessories mentioned in the technical documentation, as these, are approved by the manufacturer.
- If any other accessories or consumable materials are used, the manufacturer cannot be held responsible for the safe operation of the appliance or safety of the operating personnel.
- The warranty does not cover damages that originate from the use of accessories or consumable materials other than those specified or suggested by the manufacturer.
- The manufacturer becomes responsible for the reliability and function only if:
 - Installation, new settings, amendments, extensions and repairs are performed by the manufacturer or his representative, or a service company approved by the manufacturer,
 - The appliance is used in accordance with the installation, operation and maintenance manual.
- After its printing, the installation, operation and maintenance manual corresponds with the version of the appliance and the country according to the relevant safety and technical standards. All wiring-diagrams, methods and names in the manual are strictly copyright. They may not be reproduced in any way without prior permission of the manufacturer.

2.2. General safety warnings

The manufacturer has designed and developed the appliance so that damages are avoided when the chosen model is used for its correct intended purpose. Please take note of the following safety measures to avoid accidents, injury and damages.

- When operating the appliance, it is important to respect the laws and regional valid regulations at the place of use. The users of our equipment are responsible for the observance of all the regulations within the UK.
- Original packaging should be kept for the return of the appliance. Only original packaging ensures optimal protection during transport. If it is necessary to return the appliance during the warranty period, the manufacturer is not liable for damages caused by improper packaging.
- Before using the appliance everyday, it is important that the user convince him/herself on the due function and safe condition of the appliance and all equipment connected to it.
- The user must fully familiarise him/herself with the workings of the appliance.
- Whilst the appliance is in operation, and a technical failure occurs, the user is obliged to inform their supplier immediately.
- The appliance is not designed for operation in areas where explosive gases and flammable liquids are present.

2.3. Safety warnings on protection against electrical current

- This appliance must be connected to earth.
- Prior to connection of the appliance to the power supply, check that the voltage and frequency stated on the appliance are correct.
- Prior to switching on, check for possible damage to the appliance, suction/exhaust pipe-work system, pneumatic and electrical system. Replacing any damaged parts or wires immediately.
- Immediately disconnect the appliance from the power supply (remove the plug) if a technical failure occurs.
- Before commencing any repairs and/or maintenance, ensure that:
 - Mains plug is removed from the socket
- The appliance can only be installed by an approved qualified engineer.

3. CAUTIONS AND SYMBOLS

The following labels and symbols can be found in the installation, operation and maintenance manual, and on the appliance and packaging. Their purpose is to outline extra important information.:

	Information, instructions and cautions for the prevention of damage to health or materials
	Caution! Dangerous electric voltage
	Read the user manual!
	CE mark of compliance
	Caution! Hot surface
	Compressor is remote-controlled and may start without warning
	Earth (ground) connection
	Terminal for ground connection
	Fuse
	Alternating current
	Handling mark on package – Fragile, handle with care
	Handling mark on package – This way up (vertical position of cargo)
	Handling mark on package – Protect against moisture
	Handling mark on package – Temperature during storage and transport
	Handling mark on package – Limited stacking
	Danger of biological hazard

4. STORAGE AND TRANSPORT CONDITIONS

The manufacturer provides sufficient protection of the appliance when it leaves the factory. Therefore the product is protected against damage during storage and transport

Caution! For transport, always use the original packaging and, transport the appliance in the upright position, using secure fixings.

Keep dry. Protect appliance against humidity and extreme temperatures during storage and transport. Appliance in the original packaging must be stored in warm, dry and dust-free areas.

Keep packaging material if possible. If not, please dispose of the packaging material in an environmentally friendly way. Cardboard can be recycled.

5. TECHNICAL DATA

	DO 2.1 (*T)	DO 2.1 Z(*T)	DO 2.1-10 (*T)	DO 2.1-10 Z(*T)
Rated voltage / frequency V / Hz	230 / 50 230 / 60 (*)			
Performance of suction unit under-pressure 5kPa ltr.min ⁻¹	800			
Min. under-pressure of suction unit kPa	12			
Appliance current A	2.6			
Noise level dB(A)	≤ 46	≤ 58	≤ 46	≤ 58
Duty cycle	permanent - S 1			
Dimensions of appliance w x d x h mm	380x525x575	290x430x350	420x525x620	330x430x350
Dimensions of appliance in cardboard w x d x h mm	410x555x650	350x460x450	490x555x780	350x460x450
Weight of appliance Kg	27	15	29	15
Weight of appliance in cardboard Kg	29	18	31	18
Version according to EN 60 601-1	Appliance of B type - class I.			

Notes: (*) – Rated voltage / frequency by special request
(**) – by special request

Climatic conditions during storage and transport

Temperature: -25°C to +55°C. 24 h to +70°C
Relative air humidity: 10% to 90 % (no condensation)

Climatic conditions of operation

Temperature: +5°C to +40°C
Relative air humidity: ≤ 70%

6. PRODUCT DESCRIPTION

6.1. Model variations and their uses

Dental suction unit DO 2.1 - is the source of vacuum to provide suction in the dental surgery – aspirates waste from the patients mouth - via a tip connected to the cannule of the suction hose, through the pneumatic control module and down to the separator, where the water and foreign matter is left behind, leaving only air to be drawn (sucked) to the suction unit and exhausted away from the surgery's environment.

Dental suction unit is composed of an aspirator unit with a base, cooling fan, switching block and an insulated soundproof box. Operation of the suction unit is controlled via (24V/0.1A DC/AC) connection at the chair or via a switch on (model „T“) at the suction unit. Cooling fan draws air into the box and circulates it around the suction unit forcing air out through ventilated slots in the rear.

Dental suction units are made accordingly for the purpose in the following versions:

DO 2.1, DO 2.1-10 – stationary dental suction unit in a soundproof box – it serves as the source of vacuum for the dental chair fitted with a pneumatic control module. The suction unit will start automatically after receiving a signal from the chair – by a 24V AC/DC connection between the chair and suction unit. Soundproof box is provided for silent operation.

DO 2.1 T, DO 2.1-10 T – model as at DO 2.1 above, switch from unit controls suction pump by voltage from module „T“.

DO 2.1 Z, DO 2.1-10 Z – stationary dental suction unit without the soundproof box – it serves as the source of vacuum for the dental chair fitted with a pneumatic control module. The suction unit will start automatically after receiving a signal from the chair - by a 24V AC/DC connection between the chair and suction unit.

DO 2.1 TZ, DO 2.1-10 TZ – model as at DO 2.1 Z above, switch from unit controls suction pump by voltage from module „T“.

Dental suction units above designated as - DO 2.1-10, DO 2.1-10 T - are different from the other ones only by their dimensions and weight, so that they complied with the design of compressors - DK50-10 S type.

7. DESCRIPTION OF FUNCTION

Suction pump (1) (Fig.1) creates vacuum in the pneumatic control module thus it aspirates foreign matters away from the working area of the physician and sucks them into the separator at the dental chair. Only clean air continues past the separator through the pump of the suction unit, and then, is pushed through a supplementary silencer (6) and continues through the output hose and vented to the environment outside of the building. Suction unit is switched on by means of a switching block (3) after receiving a signal from the dental chair (voltage 24V AC/DC) or by switching-on at the suction unit (switch - model „T“). Suction pump is mounted above the cooling fan (4) which is fixed to the base (5). The cooling fan draws air into the insulated soundproof metal box (2), which stands on feet to allow the air in, and vented slots in the rear to allow the air to blow through.

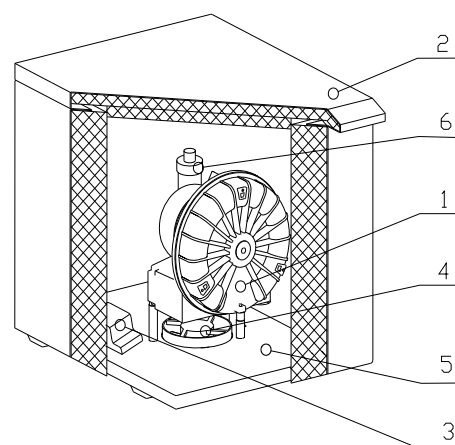
Fig.1

Make sure that nothing impedes the free flow of air under and around the compressor. Never cover the hot air outlet on the top back side of the case.

If placing the appliance onto a soft floor, such as carpet, it is important to create a gap between the base and floor or the box and floor, e.g. underpin the footings with hard pads. The cooling fan needs a gap to draw air inside the box.

INSTALLATION**8. USE**

- The appliance must be installed and operated in a dry, well ventilated and dust-free area where ambient temperature is within the range of +5°C to +40°C and relative air humidity does not exceed 70%. The appliance must be installed so that it is accessible at all times to operate and maintain. Please ensure the appliance label is accessible also.
- The appliance must stand on a flat, sufficiently stable base. See paragraph 5 (Technical data) when positioning or lifting the appliance.
- Aspirators must not be exposed to external environments. They must not be in operation in moist or wet environments. Furthermore, it is forbidden to use them in premises, where there is the presence of explosive gases or flammable liquids.
- Prior to installation of the aspirator to medical equipment, it is important to ensure that the medium available meet, the requirements fit for the purpose of use. With regards to this, respect the technical data provided. Classification and evaluation of compliance should be carried out at the installation stage.
- For any other use beyond the above scope will be considered unfit for the aspirator's purpose, and therefore, the manufacturer is not responsible for damages resulting from them. Risk is borne exclusively by the operator/user.

9. INSTALLATION

Only qualified personnel can install this appliance and by obligation, train all operating personnel on the correct use and routine maintenance. Installation and training of all operators shall be confirmed by their signature on the certificate of installation.

Prior to installation, ensure the appliance is clear of all the transport packaging and fixings to avoid any risk of damage to the product.

Caution! Contact with the appliance is dangerous. High temperatures occur during and after use. Beware of hot surfaces and the danger of fire if, contact is made by flammable objects

At a predetermined place, position the suction unit on the base (Fig.2) and remove all transport packaging and fixings. Connect both hoses to their relative inlet and outlet ports of the pump. Prepare the required electrical connection between the pneumatic control module and suction unit as below:

Connections under floor – suitably shorten both hoses and connect them to the prepared pipes protruding up through the hole in the base. Connect the 24V signal wire from the dental chair to the terminals – 1 & 2 – and connect the power cord to terminals - PE & U & N - inside the switching block.

Connection above floor – direct power cord and both hoses to the rear of the base. Thread the 24V signal wire from dental chair through the rear hole of the box and connect to the terminals in the switching block. Position the box to face the right direction, and place over the suction unit whilst at the same time, thread both hoses and power cord through the rear hole of the box. When positioned correctly, plug the power cord in the electrical socket.

NOTE:

- 1 - Outline of the case
- 2 - Outline of the base
- 3 - Front of the case
- 4 - Suction pump control voltage (2X x 0.75)
- 5 - Suction pump - IN
- 6 - Suction pump - OUT
- 7 - Supply voltage - 230V/50(60) Hz, (3G x 0.75 x 3000)

All dimensions are in millimetres

Distance of box from wall - min. 80 mm

Outline dimensions for DO2.1-10 are listed in brackets

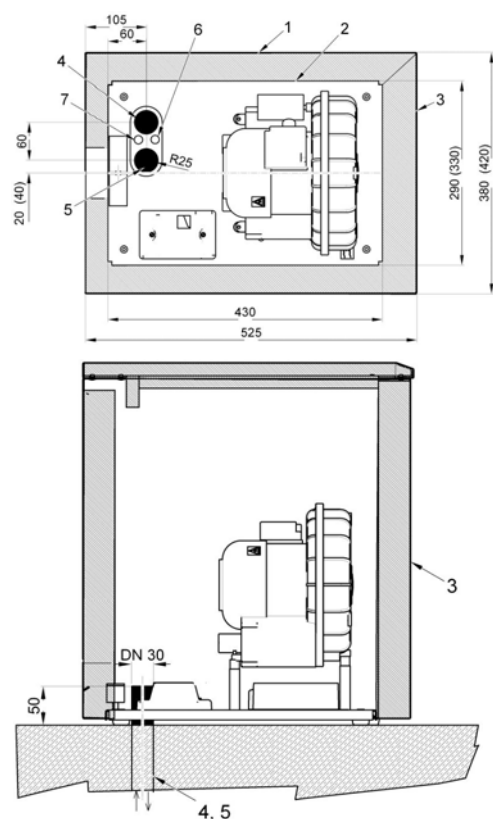
Fig.2

9.1. Vacuum Connection

Two hoses, one for suction (1) and one for exhaust (2) are connected to the suction pump (Fig.3). The hoses are inlet (1) and outlet (2) of the pump. They protrude through the rear hole of the box, or they are connected to the pipe-work under the floor. Suction (vacuum) hose is connected to the separator/pneumatic control module at the dental chair, and exhaust hose is vented away from the surgery to the environment outside of the building.

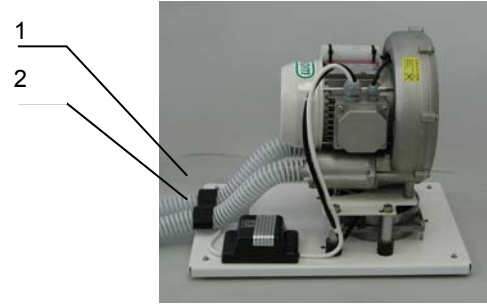
Fig.3

9.2. Electrical connection

Plug the electrical cord into the mains.

The appliance is equipped with a grounded plug. Make sure this connection complies with local electrical codes. The mains voltage and frequency must comply with the data stated on the appliance label.

- Keep the socket easily accessible to ensure that in an emergency the appliance can be safely disconnected from the mains.
- Connection to the power distribution box must be max.16 A.
- Power between the suction unit and pneumatic control module on the dental chair, is 24V provided by wires connecting to a PCB in the switching block (CP1).




If any electrical cord or air hose is damaged it must be replaced immediately.

10. WIRING DIAGRAM

230 V
ELECTRIC OBJECT OF
CLASS I
B TYPE

Mark	Name
M1	Suction pump
EV1	Fan
CP1	Switching block

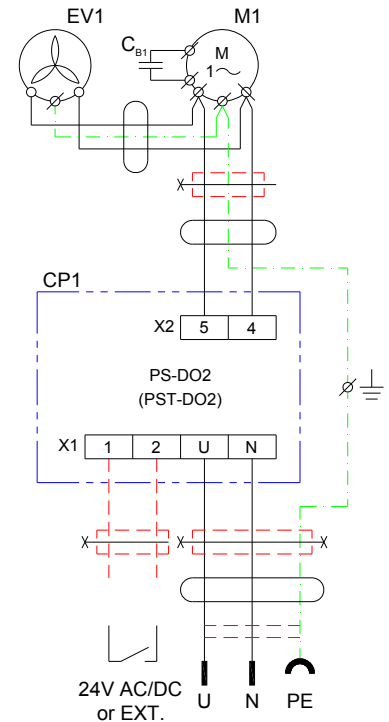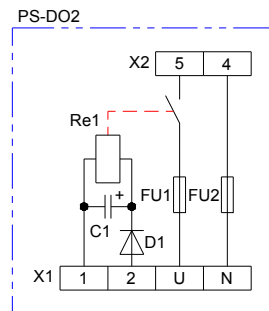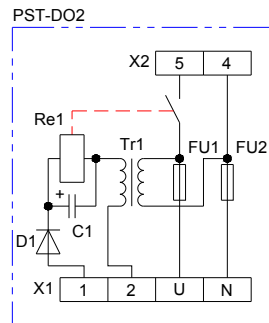11. FIRST OPERATION

- Make sure that all stabilizers used during transport were removed.
- Check all connections of the vacuum and exhaust pipe-work system.
- Check the suction unit is switched on at the plug.
- Check the suction unit switches on at the dentist's pneumatic control module.

OPERATION

In case of emergency, disconnect the suction unit from the mains (pull out the mains plug).

Parts of the suction unit can have hot surface areas. There is the danger of scorching if contact is made.

The suction unit operates after receiving a signal from the dental chair. The cooling fan and suction pump always work together.

12. SWITCHING SUCTION UNIT ON

When the suction unit is switched on at the plug, power is provided to the switching block, and puts the aspirator - on stand-by mode - waiting for a signal from the dental chair. When a signal is received, the suction unit operates and creates a vacuum in the inlet path until a signal interrupts its operation, and puts the suction unit back - on stand-by mode.

MAINTENANCE**13. MAINTENANCE SCHEDULE****Notice!**

The operating entity is obliged to ensure that all tests of the equipment are carried out repeatedly at least once within every 24 months (EN 62353) or in intervals as specified by the applicable national legal regulations. A report must be prepared on the results of the tests (e.g.: according to EN 62353, Annex G), including the measurement methods used.

Required maintenance	See document	Frequency	Performed by
• Separator - working correctly (suction path free of liquids – dry)	Documentation of dental unit	monthly	user
• Full service of suction system	Service documentation	yearly	qualified engineer
• Perform “Repeated Test” according to EN 62353	chap.13	1 x 2 years	qualified engineer

14. MAINTENANCE

Repair work beyond normal maintenance can be performed only by qualified personnel or the manufacturer’s representative.
Use only spareparts and accessories approved by the manufacturer.

Prior to any maintenance or repair work, switch off the compressor and disconnect it from the mains (pull out the mains plug).

Prior to any maintenance or repairs, the soundproof box must be removed.

The suction unit is designed and manufactured to keep maintenance to a minimum.

15. STORAGE

If the appliance is not to be used for a prolonged period of time, the manufacturer recommends operating the entire aspirating system for approximately 10 minutes, with an unrestricted flow of air (without cannules). Then disconnect the appliance from the mains. Cover the inlet and outlet ports with the plastic plugs that were supplied with the unit

16. DISPOSING OF THE APPLIANCE

- Disconnect appliance from the mains.
- Disconnect appliance from pipe-work at the inlet and outlet ports.
- To follow the rules of personal hygiene for works with contaminated material.
- To separate, label, packing and providing for decontamination of contaminated parts by course of national regulations.
- Liquidate dental suction unit according to local effective statutes.
- This appliance has no negative impact on the environment when scrapped due to failure.

Inside parts of unit can be contaminated with biological material by reason incorrect using. Before clearing and waste disposal pass on special institution for decontamination.

17. REPAIR SERVICE

Guaranteed and post-guarantee repairs must be done by the manufacturer, its authorized representative, or service personnel approved by the supplier. The manufacturer reserves the right to make changes to the appliance without notice. Any changes made will not affect the functional properties of the appliance.

18. SOLVING PROBLEMS

Prior to any inspection or repair work to the appliance, disconnect from the power supply – Switch off, remove plug from the mains socket.

The following must only be performed by trained engineers approved by the, supplier or manufacturer.

In case of repair of parts of device which might be contaminated please follow bellow mentioned instruction:

To follow the rules of personal hygiene for works with contaminated material. To separate, label, packing and providing for decontamination of contaminated parts by course of national regulations.

To perform the repair of damaged parts

FAILURE	POSSIBLE CAUSE	REMEDY
Suction unit does not work	No voltage in the socket Blown fuse in switching block Faulty power cord No presence of control voltage Suction unit is/was flooded with liquid Damaged windings of motor, damaged thermal protection Faulty capacitor	Switch on circuit breaker, replace fuse Replace faulty fuse Replace faulty cord Check power cord Check signal from dental chair Service/repair suction unit, adhere to rules on work with infectious material Replace motor Replace condenser / capacitor
Suction unit is noisy	Object in intake path Suction unit is/was flooded with liquid	Remove foreign object Service/repair suction unit, adhere to rules on work with infectious material
Suction unit aspirates poorly	Leaks in the inlet path Object in inlet/outlet paths	Cure leaks in the inlet path, remove foreign object

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