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Spec. 9391-003-66884762-2010

PURPOSE

Zinc-phosphate dental cement "Unifas-2" (hereinafter – cement "Unifas-2") is used:

- for fixing inserts, pinned teeth, crowns and bridge prostheses;
- for fixing porcelain crowns, metal-ceramic crowns and prostheses;
- for lining under other filling materials;
- for filling tooth root canals;
- for filling teeth to be crowned.

FEATURES

Cement "Unifas-2" is a unified zinc-phosphate cement, has sufficient adhesion, high physical and mechanical strength and chemical resistance, promotes the regeneration of tooth tissues, prevents the complication and development of secondary caries.

Cement "Unifas-2" consists of powder and liquid of mixing. Powder is a product of fine grinding of clinker obtained as a result of sintering a mixture of zinc, magnesium oxide, silica (quartz), bismuth oxide and ammonium molybdate with the addition of sodium fluoride and calcium hydroxide. The liquid is an orthophosphoric acid, partially neutralized with aluminum hydroxide and zinc oxide.

Cement "Unifas-2" can be produced in the shades of yellow (light-yellow, golden-yellow, etc.).

DIRECTION FOR USE

It is necessary the stored or transported at low temperatures material to stand at room temperature for at least 1 hour before applying.

Mixing should be carried out on a clean dry glass dental plate with a cement spatula. Recommended air temperature for mixing from plus 18°C to plus 23°C. At room temperature above 25°C, the plate should be kept in cold water for 2-3 minutes.

The process of mixing powder and liquid should be carried out gradually. A certain amount of powder and liquid is placed on the dental plate. First, half of the powder is mixed with the liquid, then add the remaining small portions, gradually, until a homogeneous mass is obtained. Mixing time should not exceed 90 seconds.

The necessary consistency of the cement mass for fixing the inserts, pins,

crowns and bridges, as well as for filling the tooth root canals is provided by mixing 1 dosage spoon of the powder (0.25-0.30 g) with 4 drops of liquid (0.18 - 0.20 g). The cement mass has a creamy consistency and retains plasticity on the glass for 2-2.5 minutes, and in the mouth hardens for 8 minutes.

The required consistency of the cement mass for filling teeth and lining is provided by mixing 1 dosage spoon of the powder (0.25-0.30 g) with 3 drops of liquid (0.13-0.15 g). Cement mass has a thick homogeneous consistency and retains plasticity on the glass for 1-1.5 minutes, and in the cavity of the tooth hardens for 6 minutes.

PACKAGE

Powder (jar)	- 50 g or 100 g
Liquid (flask)	- 30 g or 60g
Cap-dropper	- 1 pc.
Spoon for powder	- 1 pc.
Instruction for use	- 1 pc.
Carton box	- 1 pc.

It is possible to issue cement "Unifas-2" in a different weight and/or completeness.

STORAGE AND TRANSPORTATION

Zinc-phosphate dental cement "Unifas-2" should be stored in a dry place at a temperature of + 5°C to + 25°C.

Transport using all types of vehicles at temperatures from -30°C to +30°C.

Containers with powder and liquid must always be tightly closed. Storage of an open flask with a liquid leads to a change in its density, which affects the setting time of the cement and its strength. Do not allow foreign matter to enter the powder and liquid. The slightest contamination of the material reduces its quality.

It is necessary to closely check the compliance of a series of powder and liquid. The use of powder and liquid of different series is unacceptable.

Shelf life of zinc-phosphate dental cement "Unifas-2" is 5 years from the date of production.

Do not use after the expiry date printed on the package.

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of material.

The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product.

Responsibility for the use of the material for purposes other than those specified by the manufacturer, and material with expired shelf-life, the user's responsibility.

UTILIZATION

Residue of cement "Unifas-2" and packaging are safe for the environment and can be specified as household waste and utilized according to the rules of medical institution on the basis of current legislation.

SIDE EFFECTS

In some cases, especially sensitive patients (users) may have allergic reactions to the components of material.

About cases of identifying adverse events (incidents) not listed in the instruction for use, please inform the manufacturer.

Registration certificate № FSR 2010/09316 dated 25.04.2011.

LOT (batch number) and date of manufacture are indicated on the package.

LLC "Polymer-Dentistry"
Studencheskaya str., 52
Belgorod, 308023, Russia
tel. +7(4722) 200-555
postmaster@vladmiva.ru
www.polimerstom.ru